ACRYLUX

DENTI ARTIFICIALI IN RESINA
ACRYLIC TEETH

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
MADE IN
ITALY 

PROGETTATO E REALIZZATO IN ITALIA
DESIGNED AND MANUFACTURED IN ITALY

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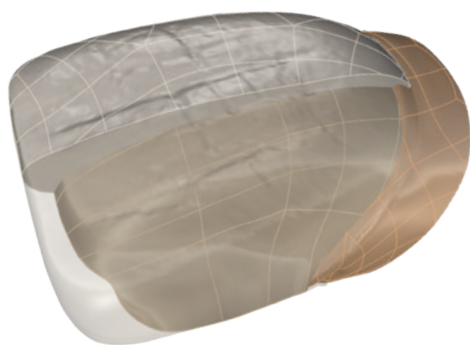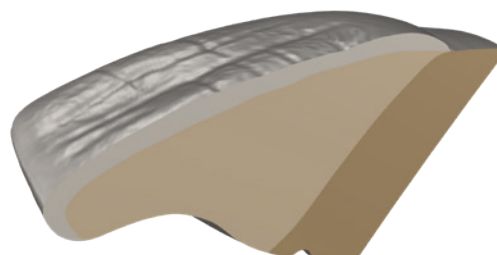
High aesthetical effect due to the 3 layers of acrylic resin and due to the crown cut of incisal edges. The wide assortment of shapes and shades make it adaptable to any prosthetic need.

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
According to Williams:

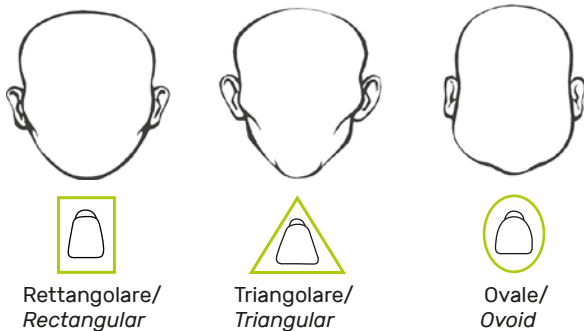The shape of the teeth is chosen according to the shape of the face.

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According to Gerber:

The shape and position of the front teeth is established by the basal line of the nose.

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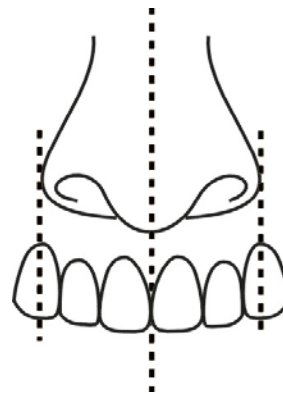
According to the shape of the alveolar crest:

If no references are provided according to the previous classifications, the shape of the upper alveolar crest can provide information on the shape of teeth to be used.

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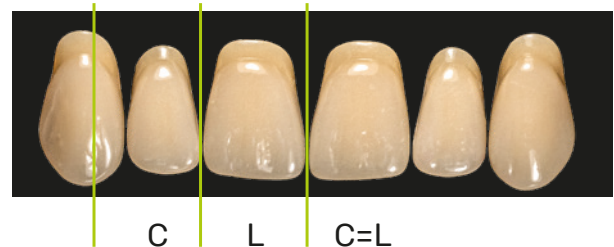
Selecting the incisor groups

Establishing the width of the anterior group according to Lee's research.

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The imaginary vertical line that touches the margins of the pinna passes through the axis of the canines.

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The central width is given by the lateral added to the mesial part of the canine.

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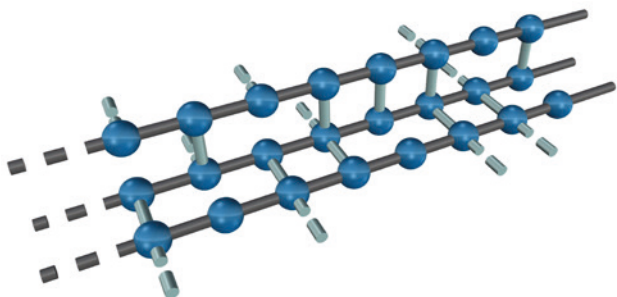
Geometrics and Anatomicals Molars

Geometrics are teeth constructed according to Starck's research for constructing bilateral balanced dentures. Cusp angle 20°.

Anatomicals are teeth characterized by anatomy accentuated to ensure modifiability for use in all mounting techniques. Cusp angle: 25°-33°.

RESINE AD ALTO PESO MOLECOLARE H.M.W. - PMMA CROSS-LINKED HIGH MOLECULAR WEIGHT RESIN H.M.W. - PMMA CROSS-LINKED

 Legame Cross Linked tra catene molecolari
Cross-Linked bond between molecular chains

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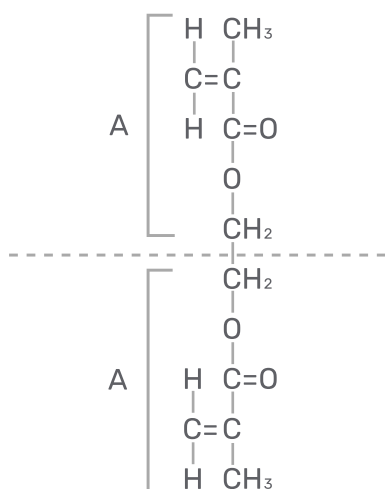
The use of high molecular weight modified polymers combined with special molecular linkers (Cross Linking) gives:

- Exceptional hardness**
- Excellent abrasion resistance**
- Color stability**
- Excellent resistance to plaque adhesion**

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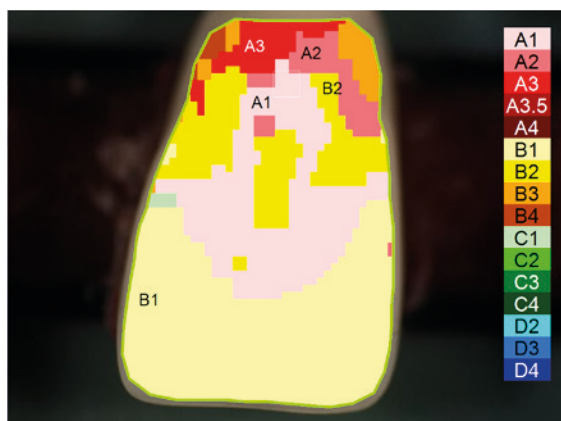
Ruthinium artificial teeth are available in a wide range of shades. The use of spectrophotometers during the control phase guarantees constant true colours.

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Fluorescence is a form of luminescence that is found in certain substances. It is a feature that is found in natural teeth which, when struck by ultraviolet rays, give off a white-bluish light. This characteristic is present in all Ruthinium teeth to give them an even more natural look.

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0090001	ACRY LUX 6 UPPER
0090002	ACRY LUX 6 LOWER
0090003	ACRY LUX 8 UPPER
0090004	ACRY LUX 8 LOWER
0100001	ACRY LUX SET 28
0110001	SHADE GUIDE ACRY LUX
0110002	LIVE MOULD CHART ACRY LUX
0090009	ACRY LUX V. 6 UPPER
0090010	ACRY LUX V. 6 LOWER
0090011	ACRY LUX V. 8 UPPER
0090012	ACRY LUX V. 8 LOWER
0090013	ACRY LUX V. SET 28
0110003	SHADE GUIDE ACRY LUX V.
0110004	LIVE MOULD CHART ACRY LUX V.

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